

# Logan Health Lab Outpatient Test Collection Aid

Logan Health Laboratory | 752-1737 | <https://logan.testcatalog.org>

## LEGEND:

Mayo Testing (F) - Fasting  
 \* See Test Catalog or Special Collection Times  
 A - Ambient R - Refrigerate Fr-Frozen

Always refer to the Logan Health Test Catalog for specific test information

Test Ordered	Meditech Mnemonic	Standard Tubes mL Volume	Tube Color Preferred	Stability	Instructions
Amylase Level	AMY.1	1	Gold	R	Spin Down within 4 hours of draw
ANA, Screen	ANA2	1.5	Gold	21d - R	<b>Send out test, must arrive to the main lab by 1400, M-F.</b>
BMP-Basic Metabolic Panel (F)	BMP.1	1	Gold	R	Spin Down within 1 hours of draw
CBC Diff or CBC No Diff	CBC or CBC ND	1	Lavender	24 hrs - A	<b>DO NOT Centrifuge</b> - 24 hrs Ambient, 48hrs Refrigerated MAX
CMP-Comprehensive Metabolic Panel* (F)	CMP.1	1	Gold	R	Spin Down within 1 hours of draw
Copper* (See Test Catalog)	COPPER	0.5	Royal Blue, EDTA		Allow specimen to clot for 30 minutes, centrifuge, serum separated from cellular fraction
Cortisol, Fasting* (special collection times)	COR	1	Gold	R	<b>NO BIOTIN or Vitamins 24hrs prior</b>   Spin Down within 4 hours of draw
Total Creatine Kinase	CK.1	1	Green-Lithium Hep	R	Spin Down within 4 hours of draw
CRP Quant	CRP.Q	1	Gold	R	Spin Down within 4 hours of draw
D-Dimer* (See Test Catalog)	DD	Full	Light Blue	2 hrs - non spun	< 2 hr arrival or > 2 hrs, spin 10 minutes at 3500 rpm, remove plasma place in clean plastic vial
Digoxin	DIG.1	1	Gold	R	Spin Down within 4 hours of draw
Dilantin	DIL	2	Gold	R	Spin Down within 4 hours of draw
Direct Coombs	DAT.1	6	Pink, EDTA	24 hrs - R	Specimen must arrive within 24 hours
ESR (Sed Rate)	ESR	1	Lavender	4hrs-A   24hrs-R	<b>DO NOT Centrifuge</b> - Specimen must arrive withing 4 hrs or 24 hrs refrigerated
Estradiol	ESTRADIOL	2	Gold	48 hours - R	<b>NO BIOTIN or Vitamins 24hrs prior</b>   Spin Down within 4 hours of draw
Ferritin	FER	1	Gold	R	<b>NO BIOTIN or Vitamins 24hrs prior</b>   Spin Down within 4 hours of draw
Fibrinogen	FIB	Full	Light Blue	2 hrs - A	Full Tube - Specimen must arrive within 2 hours of draw
Folate (F)	Folate	2	Gold	R	<b>NO BIOTIN or Vitamins 24hrs prior</b>   Spin Down within 4 hours of draw
Follicle Stimulating Hormone	FSH	2	Gold	R	<b>NO BIOTIN or Vitamins 24hrs prior</b>   Spin Down within 4 hours of draw
Triiodothyronine (T3), Free	T3F.1	1	Gold	R	Spin Down within 4 hours of draw
Free T4	T4F	2	Gold	7d - R	Spin Down within 4 hours of draw
GGT-Gamma Glutamyl Transferese	GGT.1	1	Gold	7d - R	Spin Down within 4 hours of draw
Glycohemoglobin A1C	HGBA1C.1	1	Lavender	48 hrs - R	<b>DO NOT Centrifuge</b> - Whole Blood
HCG Qualitative	PREG	1	Gold	48 hrs - R	Must arrive within 48 hrs - Spin Down within 2 hours of draw
HCG Quant	HCGQ1	2	Gold	48 hrs - R	<b>NO BIOTIN or Vitamins 24hrs prior</b>   Spin Down within 4 hours of draw
Hepatitis C Virus Antibody	HCVAB.1	1	Gold	48 hrs - R	Spin Down within 8 hours of draw
Hep B Surface Antigen	HBSAG.1	1	Gold	48 hrs - R	Spin Down within 8 hours of draw
Hepatitis B Surface Antibody	HBSAB.1	1	Gold	7d - R	Spin Down within 8 hours of draw
HIV, Type 1 & 2, Antibody Screen	HIV.1	1	Gold	R	Spin Down within 4 hours of draw
Iron Profile	IRONPR	1	Gold	R	Spin Down within 4 hours of draw

# Logan Health Lab Outpatient Test Collection Aid

Logan Health Laboratory | 752-1737 | <https://logan.testcatalog.org>

## LEGEND:

Mayo Testing (F) - Fasting  
 \* See Test Catalog or Special Collection Times  
 A - Ambient R - Refrigerate Fr-Frozen

Always refer to the Logan Health Test Catalog for specific test information

Test Ordered	Meditech Mnemonic	Standard Tubes mL Volume	Tube Color Preferred	Stability	Instructions
Lactate Dehydrogenase - LDH	LDH.2	1	Gold	48 hrs - R	Spin Down within 2 hours of draw
Lipase	LIP.1	1	Gold	R	Spin Down within 4 hours of draw
Lipid Panel (F)	LIPID	1	Gold	R	Spin down within 2 hours of draw
Liver Panel	LIVER.1	1	Gold	R	Spin Down within 4 hours of draw
Magnesium	MG.1	1	Gold	R	Spin Down within 4 hours of draw
Mumps Virus IGG Antibody	MUMPSIGG	1	Gold	8 hrs - A	Spin Down within 2 hours of draw
Parathyroid Hormone, Intact	PTH.1	2	Gold	48 hrs - R	<b>NO BIOTIN or Vitamins 24hrs prior</b>   Spin Down within 1 hours of draw
Phosphorus level (PO4)	PHOS.1	1	Gold	R	Spin Down within 4 hours of draw
Platelet Function Analysis* (See Test Catalog)	PFA	1.8	Light Blue	2 hrs - non spun	<b>DO NOT Centrifuge</b> - Fill Tube - No Butterflies, No needle over 21g
Progesterone	PROG	1	Gold	R	<b>NO BIOTIN or Vitamins 24hrs prior</b>   Spin Down within 4 hours of draw
Prolactin* (See Test Catalog)	PRL	1	Gold	R	<b>NO BIOTIN or Vitamins 24hrs prior</b>   Spin Down within 4 hours of draw
Prostate Specific Antigen	PSA	1	Gold	R	<b>NO BIOTIN or Vitamins 24hrs prior</b>   Spin Down within 4 hours of draw
Protime/INR	KPT	1.8	Light Blue	24 hrs - A	Fill tube to capacity - send in original tube
PTT	PTTA	1.8	Light Blue	1 hr - A	Specimen must arrive within 1 hour of draw - Testing completed within 4 hours of draw
Reticulocyte (Retic) Count	RETIC	3	Lavender	24 hrs - A	<b>DO NOT Centrifuge</b> - Specimen must arrive within 8 hours
Rubella Antibody	RUB.1	1	Gold	R	Spin Down within 8 hours of draw
Rubeola (Measles) IGG Antibody	RUBEOIGG	1	Gold	8 hrs - A	Spin Down within 2 hours of draw
Syphilis IGG/IGM Antibody	SYPHILIS	1	Gold	5hrs - A	<b>NO BIOTIN or Vitamins 24hrs prior</b>   Spin Down within 2 hours of draw28
Tacrolimus	TACRO	3	Lavender	14d - R	<b>DO NOT Centrifuge</b> - Draw immediately before scheduled dose -send in orignal tube
TB Quantiferon Gold ( 4 tube set ) *	TBQFT	1 (each tube)	TB Set	14 hrs - A	<b>DO NOT Centrifuge</b> - Specimen must arrive within 14 hours of draw
Testosterone, Total	TES	1	Red	14d - R	<b>Centrifuge if you are going to Aliquot Serum Specimen</b>
Testosterone, Total and Free	TES.FR	1	Red	14d - R	<b>Centrifuge if you are going to Aliquot Serum Specimen</b>
TSH	TSH	2	Gold	R	<b>NO BIOTIN or Vitamins 24hrs prior</b>   Spin Down within 4 hours of draw
TSH Reflex	TSHR	2	Gold	R	<b>NO BIOTIN or Vitamins 24hrs prior</b>   Spin Down within 4 hours of draw
Uric Acid* (See Test Catalog)	URIC.1	1.5	Gold	R	Spin Down within 4 hours of draw
Valproic Acid	VAL	1	Gold	R	Draw prior to next dose   Spin Down within 4 hours of draw
Varicella- Zoster, IGG	VZG	1	Gold	48 hrs - R	Allow blood to clot - once clotted immediately centrifuge
Vitamin B 12	VITB12	2	Gold	R	<b>NO BIOTIN or Vitamins 24hrs prior</b>   Spin Down within 4 hours of draw
Vit D, 25-Hydroxy	VIT D	1.5	Gold	R	<b>NO BIOTIN or Vitamins 24hrs prior</b>
Zinc* (See Test Catalog)	ZINC	1.5	Royal Blue, non-additive	28d - R	Allow specimen to clot for 30 minutes, centrifuge, serum removed within 4 hours